

## Packing Instructions for Returning the Scale–Aire Unit

We are providing this instruction sheet and packing materials to assist you in returning your unit.



**Package includes: Wooden base, Large box, small box,  
padded envelope for handpieces and tie downs**

You will need to supply the packing p-nuts, foam, or bubble wrap to complete the packing.

Start by building the boxes.

The small box is for the control unit, the large box is for the entire frame with compressor. The small box containing the control unit fits inside the large box.

Before you remove the control box, please remove the scaler insert from the scaler handpiece and the low and high-speed handpieces from their airlines.

These can be safely stored in the Fed-Ex envelope provided.


The Scale–Aire control box is held down by four wing nuts. These can easily be removed by hand.

Place the control unit upside down in its box. Arrange the air lines and footswitch in the remaining space. Pack with paper or packing p-nuts **See photo 1**



**Photo 1**

Open the height adjustment screw in the center of the “H” frame and remove the post. Remove the four wheels on the bottom of the Scale-Aire frame. These pull off. To keep the frame from moving and possibly from getting damaged during transport we suggest you fasten the frame to the wooden base. The wooden base should be placed as shown. **See photos 2,3, and 4**

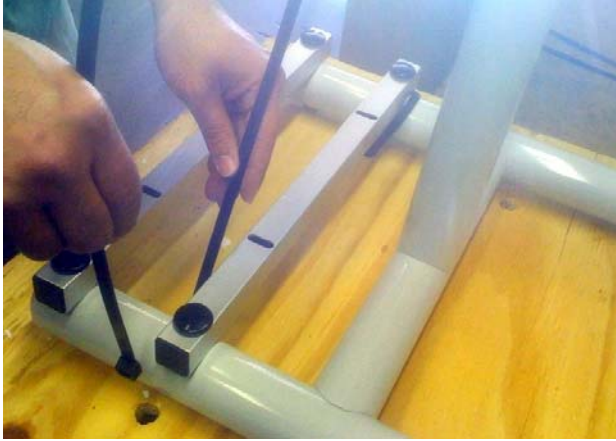

**Photo 2**



**Photo 3**



**Photo 4**



All four sides of the frame should be secured.

Place small box into the bigger box with the frame in the empty space opposite the compressor.



Reinsert the adjustable post and tighten. Note that the flange helps hold the small box in place.

Fill the box with packing p-nuts, foam and/or paper to prevent movement during shipping.

**We DO recommend you ship using UPS or Fed-Ex.**

**We DO NOT recommend you ship via United States Postal Service (USPS)**

**We DO recommend you insure the shipment for a minimum of \$3,495.00, And maintain tracking numbers. Engler Engineering Accepts NO responsibility for this shipment**

**ANY DAMAGE to uninsured equipment is CLINIC'S responsibility**

Engler Engineering Corporation  
1099 East 47th Street Hialeah, Florida 33013-2194 USA  
305-688-8581, 800-445-8581 Fax 305-685-7671 Websites: [www.englerusa.com](http://www.englerusa.com),  
[www.dynaxusa.com](http://www.dynaxusa.com), E-mail: [info@englerusa.com](mailto:info@englerusa.com)  
For manuals, brochures and our "how to" pages, [www.engler411.com](http://www.engler411.com)