

UNDERSTANDING THE FLUSH MODE

1. It is recommended that the unit be flushed at the end of each surgery. In order to flush out any debris or condensation that may build up in the A.D.S. 1000's internal lines, the A.D.S. 1000 comes with a FLUSH mode.

2. To enter the FLUSH mode perform the following:

- a. Turn OFF the A.D.S. 1000, turn the vaporizer to the off position.
- b. Place and hold your thumb over the end of the breathing circuit.
- c. Press the FILL & HOLD button and hold it down while turning ON the A.D.S. 1000.
- d. The LCD will now look like Figure 7. The A.D.S. 1000 is now in the FLUSH mode and a full 60 LPM flow of oxygen is passing through the A.D.S. 1000.

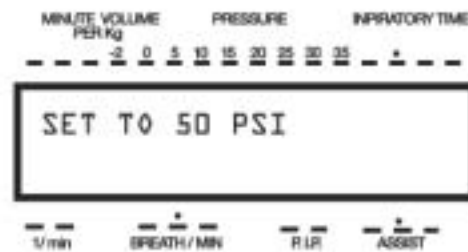


Fig.7

- e. To exit this mode, simply release the FILL & HOLD button and the unit will go through the self test.

NOTE: As a method of preventative maintenance the Flush mode should be done after every procedure. This will insure that the internal lines and valves are kept clean and dry.

UNDERSTANDING PEEP MODE

1. The A.D.S. 1000 has a built in PEEP (Positive End Expiratory Pressure) mode. To activate the PEEP mode perform the following:

- a. Turn OFF the A.D.S. 1000.
- b. Place and hold your thumb over the end of the breathing circuit.
- c. Press the ASSIST UP button and hold it down while turning ON the A.D.S. 1000.