


CARE FOR YOUR MAINTENANCE-FREE ANGLE

STERILIZATION PROCEDURES: After each prophy...

- A. Remove rubber cup.
- B. Rinse abrasive paste from head area with water.
- C. Thoroughly clean the outside of angle with disinfectant.
- D. Autoclave angle - not more than 275°F (135°C)
- E. After sterilization cycle is complete, install a new disposable rubber cup and attach angle to handpiece.

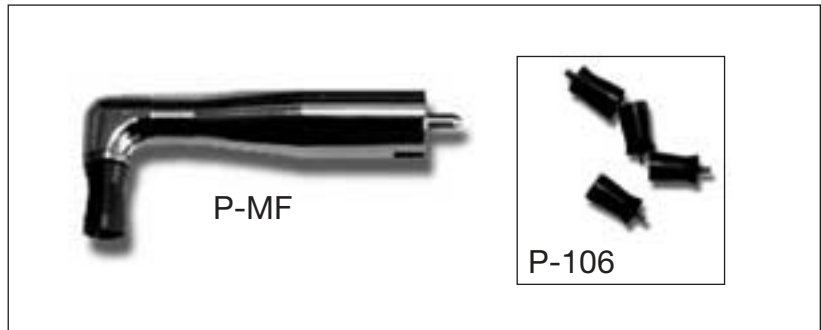

You are now ready for your next prophy.

P-MF Maintenance free prophy angle
P-106 screw on rubber cups 144/pkt

CAUTIONS AND WARNINGS:

- A. Do not attempt to disassemble.
- B. DO NOT SUBMERGE IN LIQUIDS, INCLUDING ULTRASONIC SOLUTIONS.
- C. Do not heat over 275°F (135°C).
- D. Use only Engler Care Free Prophy Rubber Cups. Other brands will not properly seal the angle, causing premature wear and voiding the warranty.
- E. Use 1 year and dispose of properly.

YOUR CARE-FREE ANGLE IS WARRANTED AGAINST DEFECTS IN MATERIAL AND WORKMANSHIP FOR 6 MONTHS. A COPY OF OUR INVOICE OR PICKING TICKET WILL BE REQUIRED AS PROOF OF PURCHASE.

WARRANTY IS VOID IF:

- A. Engler Care-Free rubber cups are not used exclusively.
- B. Sterilization procedure is not followed properly.
- C. The angle has been submerged in any liquid.
- D. The angle has been damaged or abused.
- E. Damaged due to use at high speed.