

## STACK (TRANSDUCER) INSTALLATION / REMOVAL INSTRUCTIONS


You have purchased a precision instrument  
Please handle gently- It is easily damaged

### TO REMOVE THE STACK:

- A. Unscrew the scaling tip by turning the plastic nose-cone counterclockwise.
- B. Remove tip.
- C. Pull off plastic nose-cone.  
Pull stack straight up . **DO NOT USE PLIERS!**


If you have difficulty pulling up, insert the scaling tip and pull from tip. **(IF YOU STILL CANNOT REMOVE IT CALL US AT 800-445-8581)**

- D. Do not twist or rotate stack while it is in the handpiece as it will damage the stack and / or handpiece.

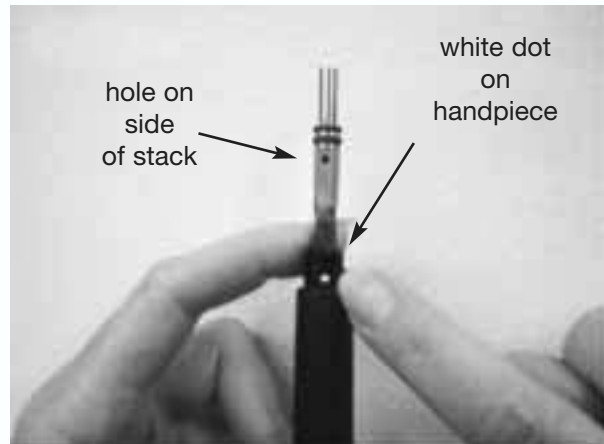

### TO INSTALL THE STACK:

- A. Carefully remove the new stack from the clear shipping tube.
- B. Locate the hole (see photo above) on side of stack. Align it with the white dot on the top of the handpiece and let the stack slide down into the handpiece.
- C. Gently push the stack down until it stops. There should be approximately 1/2 inch showing above the handpiece.
- D. Push plastic nosecone onto scaler handpiece.
- E. Place Scaling Tip into nosecone and turn nosecone clockwise. Tighten securely by hand. **DO NOT USE TIP WRENCH.**

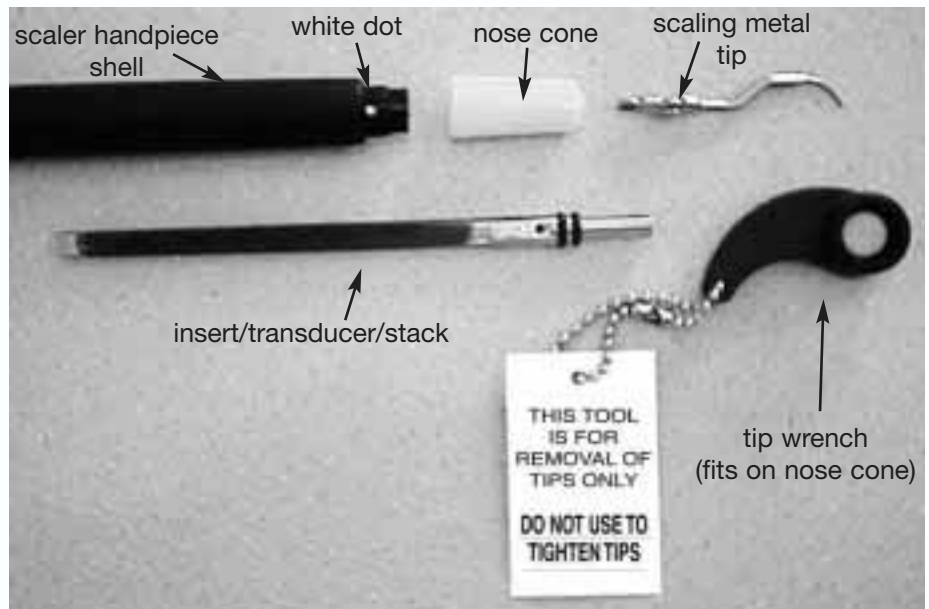

**NOTE:** Twisting the nosecone or tip with excessive force will damage the stack.

FOR FURTHER ASSISTANCE CALL CUSTOMER SERVICE  
800-445-8581 OR 305-688-8581